


Riocath[®] Rectal Tube user guide

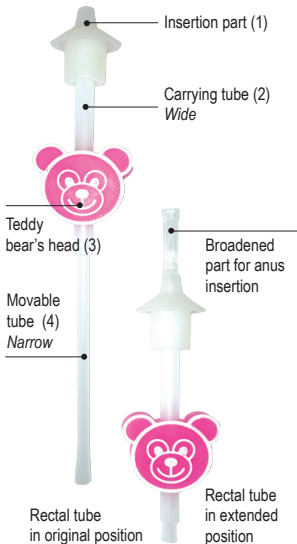

Riocath[®] rectal tube relieves bloating by releasing accumulated gas via anus.

1. Remove all the packaging and hold the Riocath[®] rectal tube by the teddy bear's head (3) with its ears pointing down (the teddy bear's head is upside down).

2. Holding the rectal tube in this direction, move it under running water and let the water run between the wide tube (2) and the narrow tube (4) located close to the teddy bear's head. (pic.1)

3. Keep on holding the rectal tube by the teddy bear's head (3) and use your other hand do hold the other (movable) tube with broadened end (4). Check the tube by pushing it in and pulling it out several times (pic.2) - it should move smoothly.

4. Lay the baby down on his or her back and rub his or her belly before inserting the rectal tube.

5. Hold the Riocath[®] Rectal Tube with your thumb at the end of narrow broadened (movable) tube (4) and both of your index and middle finger leaning on the teddy bear's ears. (pic.3)

6. Holding the baby's legs up with one hand, gently place the rectal tube near the baby's anus. Slowly push the tube with your thumb into the anus. You don't have to worry about the tube going in too deep as it only ejects the maximum of 2,5 cm which is one of the most important features of Riocath[®] Rectal Tube.

7. After inserting the tube wait for the accumulated gas to leave through it.

8. Finally, with one hand, hold the rectal tube by the teddy bear's head (3) and use your other hand to pull the narrow (movable) tube (4) into its original position.

Store in cool, dry and dark place.